## Addendum

### INFECTION CONTROL PROCEDURES

#### Reprocessing Instructions

Manufacturer: DENTSPLY Professional

Device(s): Cavitron Ultrasonic Inserts, Jet Air Polishing Insert and Nozzle, Steri-Mate Handpiece & JET-Mate Handpiece, handpiece adapters, Reservoir Bottles, and blue interface caps with sip tubes

<b>WARNINGS</b>	<ul style="list-style-type: none"> <li>• All devices must be cleaned, disinfected &amp; sterilized prior to each use. This also applies to the first use of devices supplied as non-sterile, as well as cases in which the sterile packaging has been damaged or opened.</li> <li>• High level disinfection by itself is not appropriate for these devices.</li> <li>• Thorough cleaning &amp; disinfecting are essential prerequisites for effective sterilization.</li> <li>• Only use validated methods for cleaning / disinfecting and sterilizing. Ensure equipment (washer / disinfectant and autoclave) is regularly serviced, inspected, and the validated parameters are maintained with each cycle.</li> <li>• Only use cleaning, disinfecting and ultrasonic bath solutions tested for efficacy and compatibility with the device / equipment being used (e.g. FDA certification or CE mark, accordingly).</li> <li>• Always observe all applicable legal and hygiene regulations for practice and / or hospital.</li> <li>• Always wear protective gloves, glasses and a mask when handling contaminated instruments.</li> <li>• Always use a pH neutral cleaning / disinfecting detergent or solution. Product can be damaged by alkaline and acidic detergents.</li> <li>• Read cleaning detergent / disinfecting solution instructions prior to use. Do not leave product standing in detergent or solutions.</li> <li>• Do not allow device to exceed 279°F/137°C.</li> <li>• When cleaning, cannulas and holes require particular attention.</li> <li>• An automated method (washer / disinfectant) to clean and disinfect the device is recommended. The point of use step should be performed in every case.</li> <li>• Remove all packaging prior to processing.</li> </ul>
<b>Limitations on reprocessing</b>	<ul style="list-style-type: none"> <li>• Repeated processing has minimal effect; end of life is normally determined by wear and damage due to use. See Maintenance, Inspection and Testing.</li> <li>• Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.</li> </ul>

<b>INSTRUCTIONS</b>	
<b>Point of use:</b>	<ul style="list-style-type: none"> <li>• Ultrasonic inserts must be removed from the handpiece before processing.</li> <li>• Do not allow residue or any form of contamination to dry on device.</li> <li>• Remove excess soil by wiping device with a disposable wipe under running tepid (potable) water, checking for signs of visible contamination. Dry thoroughly with a disposable wipe.</li> <li>• If using Endosonic files, remove file from PEC / PEC2 inserts &amp; discard.</li> </ul>
<b>Preparation for decontamination:</b>	<ul style="list-style-type: none"> <li>• Protect from damage.</li> <li>• Following use, device should be reprocessed as soon as is reasonably practical.</li> <li>• Device left standing wet may stain or corrode.</li> </ul>
<b>Automated Cleaning &amp; Thermal</b>	<ul style="list-style-type: none"> <li>• Use only a properly maintained, inspected, calibrated and approved washer / disinfectant according to ISO 15883 displaying an effective validation (e.g. FDA certification or CE mark, accordingly).</li> </ul>

<b>Disinfection</b>	<ul style="list-style-type: none"> <li>• Perform a cleaning program cycle in an automated instrument washer, with the recommended program cycle: <ul style="list-style-type: none"> <li>⇒ 2 minutes pre-wash/rinse in cold tap water</li> <li>⇒ 5 minutes enzymatic wash in warm water &gt; 43 °C</li> <li>⇒ 2 minutes enzymatic neutralization in hot water at 60 °C</li> <li>⇒ 10 minutes drying at 90 °C</li> </ul> </li> <li>• Perform a thermal high level disinfection cycle, with the recommended program cycle: <ul style="list-style-type: none"> <li>⇒ 5 minutes at 90 °C</li> </ul> </li> <li>• It is recommended to use sterile or low-germ (&lt; 10 CFU/ml) and endotoxin-free water (&lt; 0.25 EU/ml, e.g. high purity water HPW) for the final rinse phase</li> <li>• Follow cleaning detergent manufacturer's instructions, observing concentration rates and contact times.</li> <li>• Loading: Ensure holes / cannulas can drain. Do not exceed 279°F/137°C when drying as part of the washer / disinfecter cycle.</li> <li>• Unloading: Inspect for complete removal of visible soil, including holes / cannulas. If soil is still visible after processing, repeat cycle.</li> </ul>
<b>Cleaning: Manual</b>	<p><b>Ultrasonic Bath (Optional 1st step)</b></p> <ul style="list-style-type: none"> <li>• If soil is heavy or difficult to remove, manual pre-cleaning in an ultrasonic cleaning bath is recommended.</li> <li>• Follow manufacturer instructions for ultrasonic cleaning bath solution, observing concentration rates and contact times.</li> <li>• Adhere to the ultrasonic cleaning bath manufacturer's recommended liquid level</li> <li>• Place the device into the ultrasonic bath and cover for the indicated contact time; the instruments must be sufficiently covered.</li> </ul> <p>Rinsing</p> <ul style="list-style-type: none"> <li>• Remove the instruments from the ultrasonic cleaning bath, disposing of the used solution. Rinse solution from device by placing device under running tepid (potable) water and wiping with a disposable wipe for 30 seconds.</li> <li>• Pay special attention to rinse all holes / cannulas. Cannulas may be rinsed several times using a disposable syringe.</li> <li>• Dry device with a disposable wipe.</li> </ul> <hr/> <p>Cleaning</p> <ul style="list-style-type: none"> <li>• Follow manufacturer instructions for cleaning detergent, observing concentration rates and contact times.</li> <li>• Remove soil with cleaning detergent while wiping with a disposable wipe until free of visible soil.</li> <li>• Pay special attention to clean all holes / cannulas</li> </ul> <p>Rinsing</p> <ul style="list-style-type: none"> <li>• Remove detergent from device by placing device under running tepid (potable) water and wiping with a disposable wipe for 30 seconds.</li> <li>• Pay special attention to rinse all holes / cannulas. Cannulas may be rinsed several times using a disposable syringe Dry device with a disposable wipe.</li> </ul>
<b>Disinfection: Manual</b>	<p>Disinfecting</p> <ul style="list-style-type: none"> <li>• After cleaning thoroughly, wipe all device surfaces with a disposable wipe in combination with a bactericidal, virucidal, and fungicidal instrument disinfection solution (approved according to local regulations and used according to device disinfectant solution manufacturer's Instructions for Use), observing concentration rates and contact times.</li> <li>• Pay special attention to disinfect all holes / cannulas.</li> </ul> <p>Rinsing</p>

	<ul style="list-style-type: none"> <li>Remove disinfectant solution residue by placing the device under running tepid (potable) water, careful to follow disinfectant manufacturer's instructions.</li> <li>Pay special attention to rinse all holes / cannulas. Cannulas may be rinsed several times using a disposable syringe. Dry device with a disposable wipe.</li> </ul>
<b>Drying: Manual</b>	Dry thoroughly with a disposable wipe
<b>Maintenance, Inspection and Testing:</b>	<ul style="list-style-type: none"> <li>Inspect to ensure all visible contamination has been removed.</li> <li>Check for damage and wear: corrosion, pitting, discoloration, cracking, edges should be free of nicks. For Ultrasonic Inserts, refer to the wear guide.</li> <li>Inspect all O-rings for wear; replace worn O-rings (For Ultrasonic Inserts, refer to O-Ring Replacement Kit Instructions).</li> <li>Discard damaged or corroded product. If not visibly clean, repeat the process or safely dispose of device.</li> </ul>
<b>Packaging</b>	<ul style="list-style-type: none"> <li>Place product in a paper or paper/plastic steam sterilization pouch.</li> <li>Ensure pack is large enough to contain the product without stressing the seals.</li> <li>Product may be loaded into dedicated instrument trays or general-purpose sterilization trays.</li> </ul>
<b>Sterilization:</b>	<p>Use only validated steam vacuum autoclaves. Do not exceed 279°F/137°C</p> <p><b>Cassette Pressure Pulse Steam Sterilization</b> Full cycle: Bagged 135°C (275°F) for 3.5 minutes</p> <p><b>Pre-vacuum Steam Sterilization</b> Full cycle: Bagged 132°C (270°F) for 3 minutes</p> <p><b>Gravity Steam Displacement Sterilization</b> Full Cycle: Bagged 132°C (270°F) for 10 minutes</p> <p><b>Alternate Method:</b> Place non-bagged instruments into the steam autoclaves and run at the listed cycles.</p> <p>Dry, using the drying cycle of the autoclave.</p> <p>Note: Instruments sterilized unbagged should be used immediately.</p>
<b>Storage:</b>	To maintain sterility, instruments should remain bagged until ready for use in a dry, clean location.

<b>Additional Information:</b>	When sterilizing multiple instruments in one autoclave cycle, ensure that the sterilizer's maximum load is not exceeded.
<b>Manufacturer contact:</b>	Within the United States, call DENTSPLY Professional at 1-800-989-8826. For areas outside of the United States, contact your local DENTSPLY Professional representative.

The instructions provided have been validated by the manufacturer of the medical device as being capable of preparing a medical device for re-use. It remains the responsibility of the processor to ensure that the reprocessing is actually performed using equipment, materials and personnel in the reprocessing facility to fully achieve the desired result. This normally requires validation and routine monitoring of the process.

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